AeroChamber Plus Flow-Vu®

Anti-Static Valved Holding Chamber

MOUTHPIECE / LARGE MASK

INDICATIONS FOR USE

This product is intended to be used by patients who are under the care or treatment of a physician or licensed healthcare professional. The device is intended to be used by these patients to administer aerosolized medication from most pressurized Metered Dose Inhalers. The intended environments for use include the home, hospitals and clinics.

Distributed by:

FOREST PHARMACEUTICALS, INC.
 Subsidiary of Forest Laboratories, LLC
 St. Louis, Missouri 63045

Monaghan Medical Corporation, 5 Latour Ave.,
 Suite 1600, Plattsburgh, NY 12901

INSTRUCTIONS FOR USE

THIS PRODUCT CAN BE USED RIGHT OUT OF PACKAGE. BEFORE USE, ENSURE THESE INSTRUCTIONS AND THE INSTRUCTIONS SUPPLIED WITH THE METERED DOSE INHALER (MDI) HAVE BEEN READ.

	1	2	3	4	5	6	7
MOUTHPIECE							
	Carefully examine the product for damage, missing parts or foreign objects. Remove any foreign objects prior to use. The product should be replaced IMMEDIATELY if there are any damaged or missing parts. If necessary, use the Metered Dose Inhaler (MDI) alone until a replacement is obtained. If the patient's symptoms worsen, please seek immediate medical attention.	Remove cap(s) from the MDI and chamber (if applicable).	Shake the MDI immediately before each use as per the instructions supplied with the MDI.	Insert the MDI into the Backpiece of the chamber.	Flow-Vu® Inspiratory Flow Indicator Put mouthpiece into mouth and close lips around it to ensure an effective seal. The Flow-Vu® Inspiratory Flow Indicator (IFI) only moves if the patient has a good seal. Apply mask to face and ensure an effective seal. The Flow-Vu® Inspiratory Flow Indicator only moves if a good seal is created.	Breathe out gently and press the MDI at the beginning of a slow inhalation. Use the Flow-Vu® IFI to assist in the coordination of this step. Breathe in slowly and deeply through the mouth until a full breath has been taken. Hold breath for 5 – 10 seconds, if possible. Otherwise, keep lips tight on the mouthpiece breathing normally 2 – 3 times through the chamber after the MDI is pressed. <i>Slow down inhalation if you hear the FlowSignal® Whistle sound.</i> It means you are inhaling too quickly. Administer one (1) puff at a time. Breathe out gently and press the MDI at the beginning of a slow inhalation as the Flow-Vu® IFI moves towards you. Use the Flow-Vu® IFI to count breaths. Maintain seal for 5 – 6 breaths after the MDI is pressed. <i>Slow down inhalation if you hear the FlowSignal® Whistle sound.</i> It means you are inhaling too quickly. Administer one (1) puff at a time.	Follow instructions supplied with the MDI on how long to wait before repeating steps 3 – 6 as prescribed.
LARGE MASK							

CLEANING INSTRUCTIONS FOR MASK AND MOUTHPIECE CHAMBERS

THIS PRODUCT CAN BE USED RIGHT OUT OF THE PACKAGE AND THEN CLEANED WEEKLY.

Remove the Backpiece. To detach the Frontpiece, twist chamber as shown above. Remove the mouthpiece cap.	Soak the parts for 15 minutes in a mild solution of liquid dish detergent and lukewarm clean water. Agitate gently. Rinse parts in clean water.	Dishwasher safe under the following conditions: - Avoid heavy or other cycles that include heated dry over 158°F - Parts must be placed on the top rack ONLY - Secure product face up as pictured - Use dishwasher detergent and rinse aid - Do not boil or sterilize SEE CAUTION #1 BELOW.	To reassemble, fit the Frontpiece on the end of the chamber and twist firmly until securely locked into position. For mouthpiece models, the protective cap should always be placed on the mouthpiece when the chamber is not in use.	Center the Alignment Feature on the Backpiece with the Flow-Vu® Inspiratory Flow Indicator, as shown. Press firmly to attach the backpiece.

Notes:

- Storage and operating range 5° C - 40° C (41° F - 104° F) at 15 to 95% relative humidity.
- Product may need to be replaced after 12 months of use. Environmental conditions, storage and proper cleaning can affect product life span.
- Do not share this medical device.
- If you notice medication build-up in your chamber, wash the inside of the chamber gently with a soft cloth.
- Dishwashing with overly dirty dishes is not recommended.
- Dishwasher validated up to 158°F (70°C).

CAUTIONS:

- PRODUCT MAY BE PERMANENTLY DAMAGED IF BOILED, STERILIZED OR CLEANED IN A DISHWASHER AT TEMPERATURE ABOVE 158°F (70°C).
- Do not leave the chamber unattended with children.
- Federal (USA) law restricts the sale of this device on or by the order of a physician.

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FOREST PHARMACEUTICALS, INC.
Subsidiary of Forest Laboratories, LLC
St. Louis, Missouri 63045

Monaghan Medical Corporation, 5 Latour Ave.,
Suite 1600, Plattsburgh, NY 12901

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