


INSTRUCTIONS FOR USE

AbbVie Fitted Belt


For use with CADD-Legacy® 1400 pump

This guide pertains only to the use of the AbbVie Fitted Belt. For questions or problems, refer to these Instructions for Use or call support toll free at 1-844-386-4968.

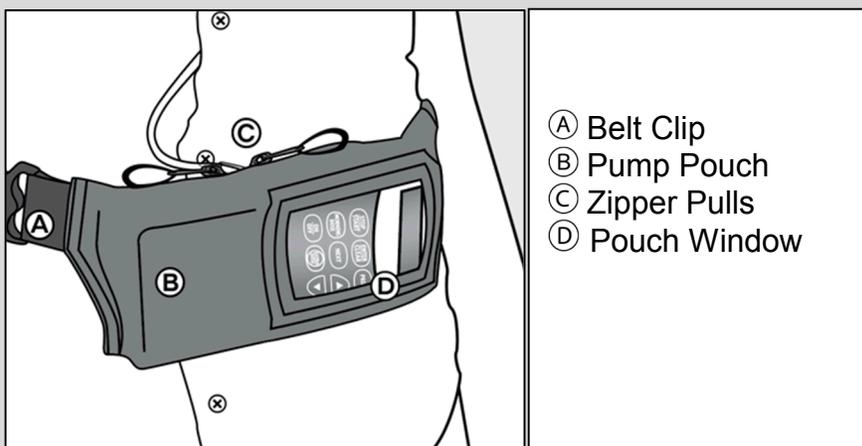

About your Fitted Belt:

The Fitted Belt is designed to carry a CADD-Legacy[®] 1400 pump. This Fitted Belt is intended for single-patient use only. The Fitted Belt is intended to be worn around the waist (see Figure i).

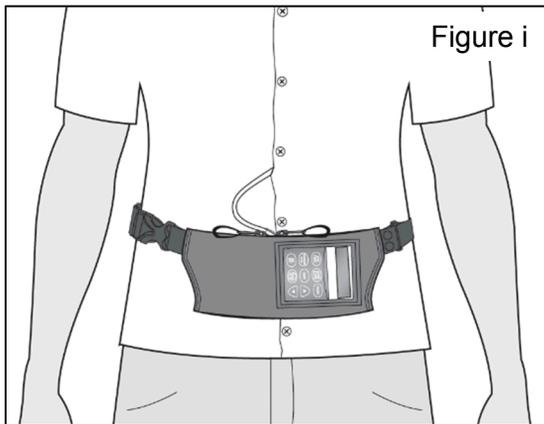

Caring for your AbbVie Fitted Belt:

Materials: 95% polyester, 5% spandex


Care: Machine wash separately in cold water, hand wash, or dry clean.
Lay flat to dry.

Disposing of your AbbVie Fitted Belt:

Dispose of the device according to your local governing ordinances and recycling plans. Otherwise contact AbbVie for help with disposal. Each device is for single patient use only and cannot be re-sold or used on multiple patients.

Precautions:

- **CAUTION:** When putting on the Fitted Belt, ensure the pump is resting on a flat surface, and will not fall (see Figure ii). **If the pump is not completely secure, it may fall and injure you or damage the pump.**

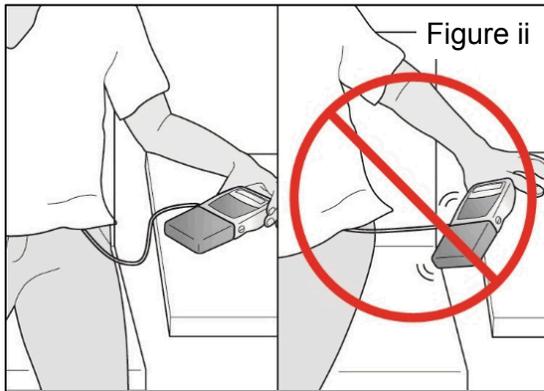


- **CAUTION:** In the event the pump is dropped and no noticeable injury is suffered, inspect the pump for damage. In the event of pump damage contact your healthcare provider.

To begin using your Fitted Belt:

1) Put on the Fitted Belt:


- **CAUTION:** Make sure that the pump is stable on a flat surface and will not fall while you put on the Fitted Belt.
 - a. Place the Fitted Belt on the front of your waist with the Pouch Window facing out and the zipper on top.
 - b. Fasten the clip (see Figure A). Listen for a snap sound.

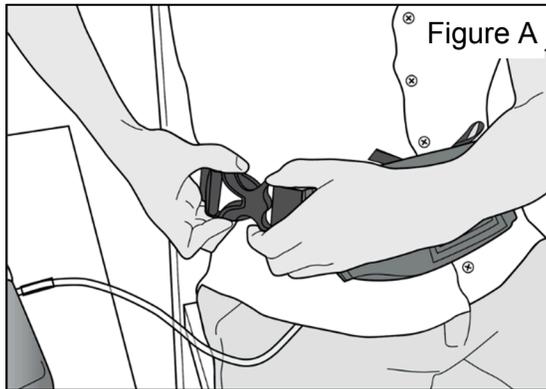

- c. Confirm that the clip is securely fastened.
- d. Confirm that the Fitted Belt remains on your waist without being held.

2) Adjust the Fitted Belt:


- a. Move the plastic slider along the Waist Strap until the Waist Strap is at the desired length (see Figure B).

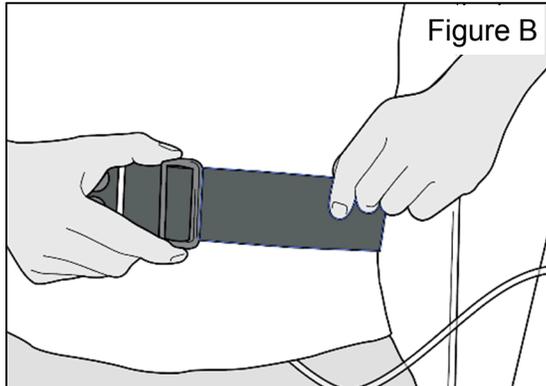

- b. Confirm that the pack is held securely against your body. The Fitted Belt should fit comfortably but not move or shift.

3) Insert the pump into the Pump Pouch:

- a. Using the two Zipper Pulls, ensure the Pump Pouch is completely open.
- b. Place the pump into the Pump Pouch horizontally with the screen on your left side and facing out (see Figure C).

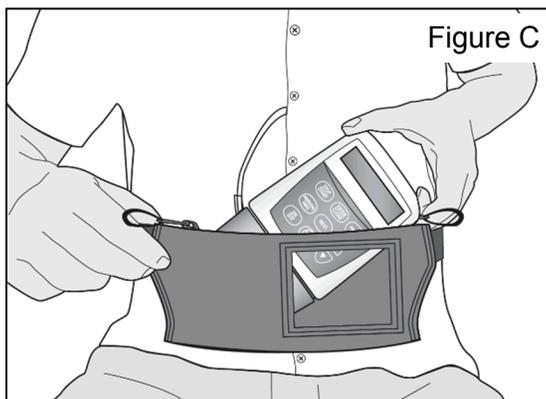


- c. Confirm that both the display and the keypad are visible through the Pouch Window.
- d. Route the tubing to exit through the center of the opening on the top of the Pump Pouch. Make sure that the tubing is not kinked.

e. Zip the Pump Pouch shut. While zipping, ensure the tubing does not become pinched.

4) Interact with the pump (when needed):

You can access the pump's keypad through the Pouch Window (see Figure D).

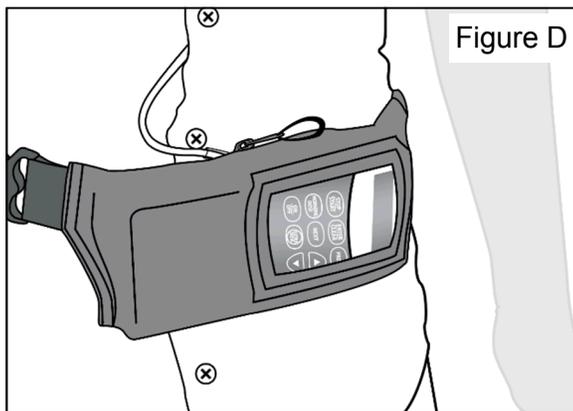

To take off your Fitted Belt:

- 1) Remove the pump from the Pump Pouch:
 - a. Pull the zippers to open the Pump Pouch.
 - b. Remove the pump from the Pump Pouch.
 - c. Place the pump on a flat surface.
- **CAUTION:** Make sure that the pump is stable on a flat surface and will not fall while you take off the Fitted Belt.
- 2) Take off the Fitted Belt:
 - a. Undo the clip.
 - b. Remove the Fitted Belt from your waist.

Explanation of Symbols (per ISO 15223-1):



Catalog number



Batch code



Manufacturer



Medical Device



Use-by date



rxabbvie.com

Consult *instructions for use* or
consult *electronic instructions for use*



Caution



0AV600-004-001



Manufactured for:
AbbVie Inc.
1 North Waukegan Road
North Chicago, IL 60064 USA



rxabbvie.com

rxabbvie.com