


INSTRUCTIONS FOR USE

AbbVie Hip Pack


For use with CADD-Legacy® 1400 pump

This guide pertains only to the use of the AbbVie Hip Pack. For questions or problems, refer to these Instructions for Use or call support toll free at 1-844-386-4968.

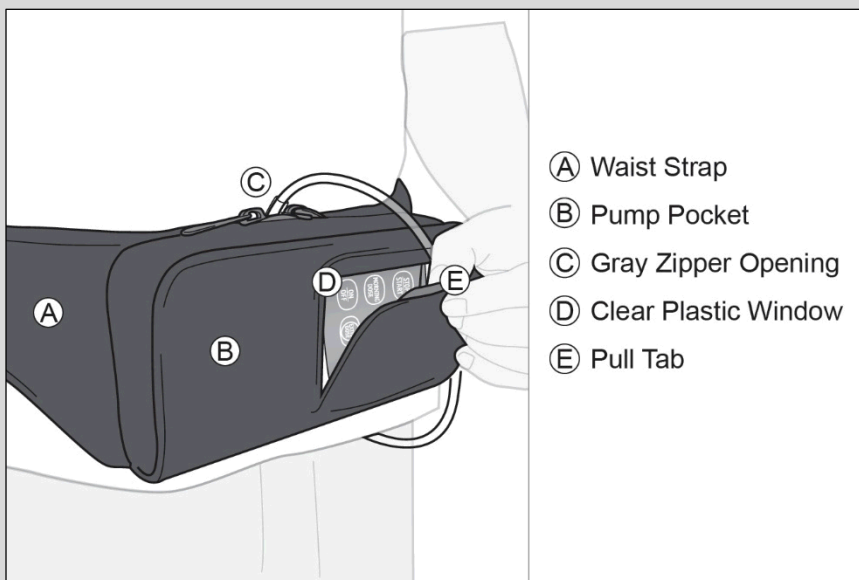

About your Hip Pack:

The Hip Pack is designed to carry a CADD-Legacy[®] 1400 pump. This Hip Pack is intended for single-patient use only. The Hip Pack is intended to be worn around the waist (see Figure i).

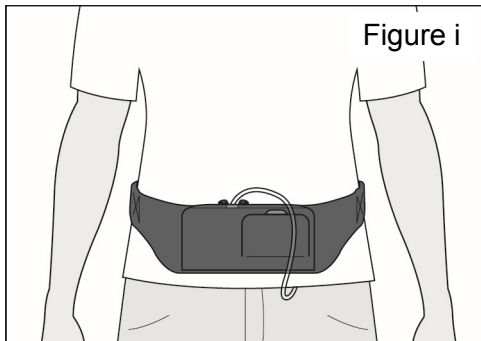


Caring for your AbbVie Hip Pack:

Care: Do not machine wash.

Precautions:

- ⚠ **CAUTION:** When putting on the Hip Pack, ensure the pump is resting on a flat surface, and will not fall (see Figure ii). **If the pump is not completely secure, it may fall and injure you or damage the pump.**

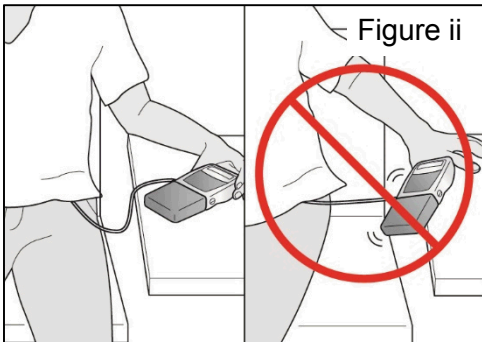


- ⚠ **CAUTION:** In the event the pump is dropped and no noticeable injury is suffered, inspect the pump for damage. In the event of pump damage contact your healthcare provider.

To begin using your Hip Pack:

1) Put on the Hip Pack:

⚠ **CAUTION:** Make sure that the pump is stable on a flat surface, and will not fall while you put on the Hip Pack.


- a. Place the Hip Pack on the front of your waist with the Pull Tab on the Pump Pocket facing out and the zipper on top.
- b. Fasten the clip (see Figure A). Listen for a snap sound.

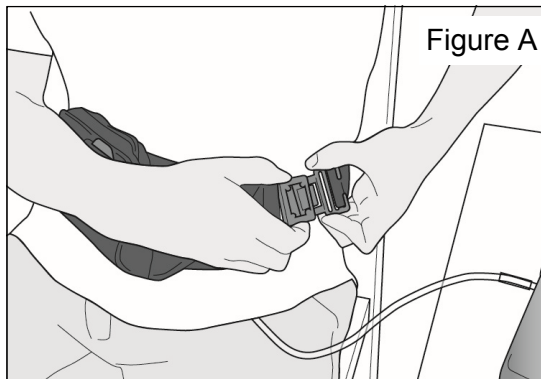

- c. Confirm that the clip is securely fastened.
- d. Confirm that the Hip Pack remains on your waist without being held.

2) Adjust the Hip Pack:


- a. Move the plastic slider along the Waist Strap until the Waist Strap is at the desired length (see Figure B).

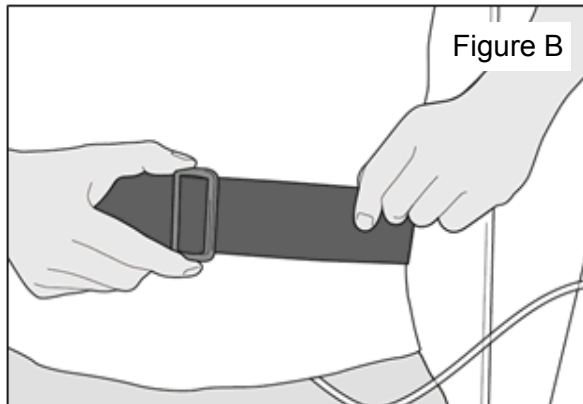

- b. Confirm that the pack is held securely against your body. The Hip Pack should fit comfortably but not move or shift.


3) Insert the pump into the Pump Pocket:

- a. Place the pump into the Pump Pocket horizontally with the screen on your left side and facing out (see Figure C).

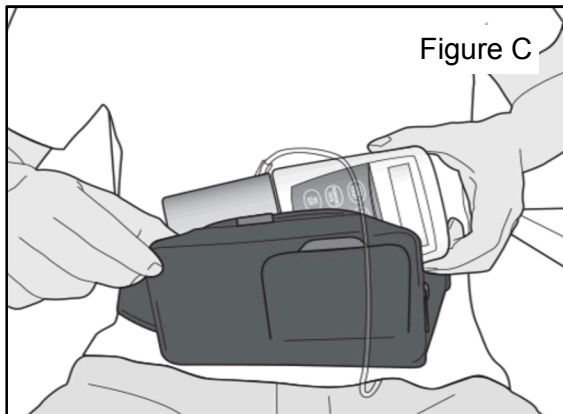

- b. Using the Pull Tab, confirm that both the display and the keypad are visible through the Clear Plastic Window.
- c. Route the tubing to exit through the Gray Zipper Opening on the top of the Pump Pocket. Make sure that the tubing is not kinked.
- d. Zip the Pump Pocket shut.

- 4) Interact with the pump (when needed):
- a. You can access the pump's keypad through the Clear Plastic Window.
 - b. Use the Pull Tab to view or cover the Clear Plastic Window (see Figure D).

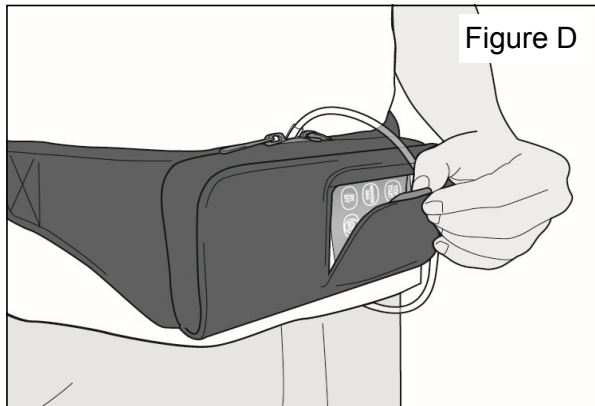

To take off your Hip Pack:

1) Remove the pump from the Pump Pocket:

⚠ **CAUTION:** Make sure that the pump is stable on a flat surface, and will not fall while you take off the Hip Pack.

- a. Pull the zippers to open the Pump Pocket.
- b. Remove the pump from the Pump Pocket.
- c. Place the pump on a flat surface.

2) Take off the Hip Pack:

- a. Undo the clip.
- b. Remove the Hip Pack from your waist.

Disposing of your Hip Pack:

Dispose of the device according to your local government ordinances and recycling plans. Otherwise contact AbbVie for help with device disposal. Each device is for single patient use only and cannot be re-sold or used on multiple patients.

Explanation of Symbols (per ISO 15223-1):



Catalog number



Batch code



Manufacturer



Medical Device



Use-by date



rxabbvie.com

Consult *instructions for use* or consult
electronic instructions for use



Caution



10637-04-01



Manufactured for:
AbbVie Inc.
1 North Waukegan Road
North Chicago, IL 60064 USA



rxabbvie.com

rxabbvie.com